 ZENIT

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Minimicroscope “Cycle”

for early diagnostics of pregnancy

By means of “Cycle” minimicroscope

one has the following possibilities:

- a woman may avoid the undesired pregnancy;
- it's possible to define the most favourable time for planning posterity;
- to define that a woman is pregnant.

Individual express-diagnostics by means of “Cycle” minimicroscope allows to quickly define favourable and unfavourable days for conception during a woman’s menstrual cycle without intervention into a human body and performing some kind of medical tests.

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The operation principle of this device is based on the change of physical properties of the saliva depending on the availability of estrogens and salts in it. Such a change is directly connected with the presence of sexual hormones in a female organism.

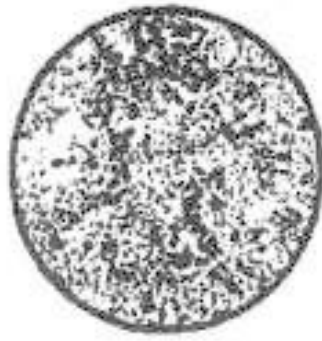
Operation order:

Apply a drop of the saliva on an object glass of a microscope. Let the sample of the saliva dry.

Then switch on an illuminator.

View the resulted image comparing it with a graphical picture – a primary standard.

The absence of "fern" picture on the viewed image of the saliva drop in days usual for its appearance testify a woman is pregnant.

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Specifications

Magnification of a magnifier	51 ^x
Period of time of an express-analysis	10 - 15 min
Power supply	2 elements of 386A, G12 type
Illumination	LED of АЛ336И

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